# CALYPSO ELECTRIC BREASTPUMP

Instructions for use | Instrucciones de uso

 ARDO

## Table of contents

1. Important safety information
2. Intended use
3. Product description
4. Start-up / Operation
5. Useful tips
6. Troubleshooting
7. Maintenance / Care / Cleaning
8. Warranty
9. Technical Information
10. Transport / Storage / Disposal

Congratulations on your purchase and thank you for choosing the gentle Calypso breastpump from Ardo.

# 1. Important safety information

## READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

When using electrical products, especially when children are present, basic safety precautions should always be followed.

### DANGER: To reduce the risk of electrocutions:

- Always unplug electrical product immediately after use.
- Do not use while bathing, showering or swimming.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not place or drop into water or other liquid.
- Do not reach for a product if it has fallen into water. Immediately unplug from electrical outlet.

MEDICAL – GENERAL MEDICAL EQUIPMENT  
AS TO ELECTRICAL SHOCK, FIRE AND  
MECHANICAL HAZARDS ONLY  
IN ACCORDANCE WITH  
ANSI/AAMI ES60601-1 (2005, C1:09, A2:2011),  
CAN/CSA C22.2 No.60601.1 (2008, R:2011)

### WARNING: To avoid damage, electric shock, injury or fire:

- Never leave product unattended when plugged into electrical outlet.
- Close supervision is necessary when this product is used near children or persons with disabilities.
- Use product only for intended use as described in this manual.
- Only use attachments recommended by the manufacturer.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- Keep cord and all attachments away from heated surfaces.
- Never use while sleeping or when drowsy.
- Do not use outdoors, or operate where aerosol (spray) products are being used where oxygen is being administered.
- Batteries should be removed from the product when using the AC.
- Use the unit with the Ardo power adapter only.
- Do not leave old batteries in the unit – danger of leakage.
- This product is intended for use by a single user only and should not be shared between users.
- Do not shorten the tubing.
- Never use while pregnant, as pumping can induce labor.
- Clean and sanitize all parts that come in contact with your breast and breast milk prior to first use.
- Do not use solvents or abrasives.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and breastshell with your finger and remove the breastshell from your breast.

# SAFE THESE INSTRUCTIONS

## 2. Intended use

The Calypso breast pump is intended to be used by lactating women to express and collect milk from their breast.

It can be used as a single pump and as a double pump. The unit is intended for indoor use only and is intended for single users.

## 3. Product description

- a Calypso breastpump
- b Pumpset (may vary with different models)
- c Tube connector
- d Control panel
- e LCD display
- f Connection opening for tube connector
- g Power adapter
- h Unit plug
- i Electrical connection socket

## 4. Start-up / Operation

### 4.1 Controls

Stand-by switch

Adjust the vacuum (V)

Adjust the cycle (C)


### 4.2 Preparing the pump for use

Outlet operation:

- Insert the unit plug (h) of the power adapter (g) into the electrical connection socket (i) of the unit (Fig. 1). The power adapter must correspond to the local electricity supply.  
Important: remove the batteries before plugging the unit into the outlet.

Battery operation:


- Release the latch (Fig. 2) and slide open the battery cover.
- Six LR 06 / AA batteries are required for use. Check the + / - mark orientation of the batteries (Fig. 3).
- Close the battery compartment cover and secure by re-engaging the latch (Fig. 4). The average working life with a medium vacuum and cycle setting is approximately 5 hours (Duracell batteries). We recommend that you use Duracell batteries.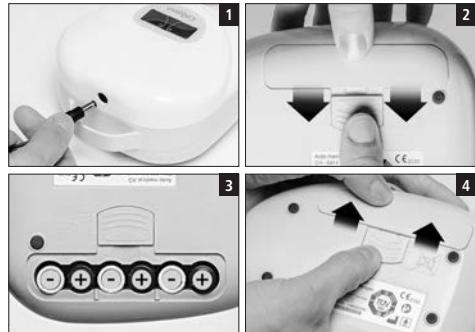

### Connecting the Pumpset

The Pumpset is an applied part of Calypso. Use of the Pumpset (b) is detailed in the separate "Ardo Pumpset" instructions for use.


- To use the single Pumpset: attach the tube connector (c) to the pump (f) (Fig. 5). The second opening of the tube connector remains closed.
- To use the double Pumpset: Open the second opening of the tube connector and attach the second Pumpset (Fig. 6).
- Attach the tube connector (c) to the pump (f) (Fig. 7).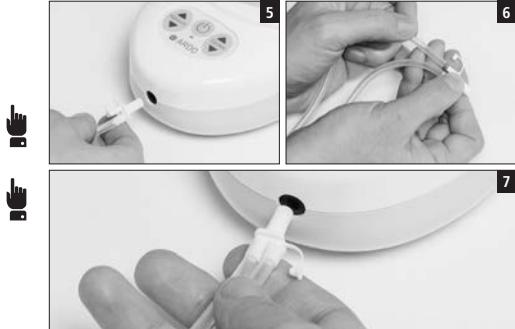

### 4.3 Operation

The vacuum (suction strength) and cycle (suction speed) can be adjusted independently of each other. Pressing the switches changes the vacuum or the cycle by one level.

- Turn on the pump by pressing the stand-by switch (Fig. 8). The green LED display will light up. Choose the settings which you find most comfortable.
- Press the up-arrow to increase the vacuum (suction strength). Press the down-arrow to reduce the vacuum (Fig. 9).
- Press the up-arrow to step-up the cycle (suction speed). Press the down-arrow to reduce the cycle (Fig. 10).

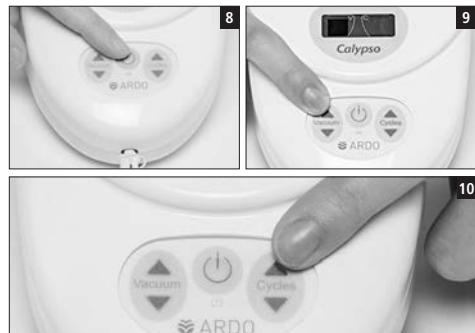## 5. Pumping

Pumping milk should never be painful.

### Starting pump

- Wash your hands thoroughly.
- Attach the pumpset to the Calypso.
- Sit down comfortably and relax.
- Centre the nipple so that, when pumping, it moves freely in the breast shell / breast shell insert.
- Fill breast shell with the breast to create an air seal.
- Switch on the pump.

### Adjusting the vacuum (suction strength)

- Regulate the vacuum by pressing the arrow keys.
- Intensify the vacuum, until you feel a very slight discomfort, and then reduce the intensity of the vacuum until pumping is comfortable.

### Adjusting the cycle (suction speed)

- Regulate the cycle by pressing the arrow keys.
- Set the cycle at the highest level which is comfortable for you.

### Encouraging the milk ejection reflex

- When babies start to breastfeed, they suckle with rapid sucking motions and low suction strength – this causes the milk to release by stimulating the milk ejection reflex. This effect can be replicated by starting with a higher cycle level (e.g. 8 bars) and selecting a weaker vacuum.
- When the milk begins to release, reduce the cycles by pressing the down arrow and strengthen the vacuum by pressing the up-arrow until you find the most comfortable level for you.

The milk ejection reflex may occur later when pumping than when breast feeding. The following measures may be helpful:

- Massage the breast before breast feeding.
- Relax and think about your baby.
- Gaze at a picture of your baby.
- Apply a warmed compress to the breast before expressing.
- Warm the cone or funnel of the breast shell before applying.

### Pumping efficiently

- Massaging the breast while using the breastpump can increase milk production, and promote better flow.
- Use a double pumpset
- For the first 10 minutes, or until no more milk is flowing, use a double pump i.e. express from both breasts at once. Afterwards, switch to a single pump. Express from each breast for five minutes while massaging gently.

## 6. Troubleshooting / error indicator

In the event of an error, the LED display flashes green and orange alternately. There are three different error-displays:

### General error (time-out)

- Switch off the unit and re-start. If the error message appears again, the unit is defective and cannot be repaired.

### Voltage too low

- Weak batteries
- Incorrect or defective power adapter

### Voltage too high

- Incorrect or defective power adapter

### Further possible errors when using the unit

- Weak or absent vacuum:  
Disconnect the Pumpset from the unit. Partly close the connector opening (f) with a finger to test the suction. However, be careful not to completely cover the opening. If suction is detectable, then the pump is working. Test the Pumpset in accordance with the separate instructions for use (only use the original Ardo Pumpset).
- The pump is not working / no data on the display:  
Check the electrical supply; if the unit still does not work it is defective.
- In the event that one of these errors occurs, and is confirmed, the unit must be brought or sent back to the point of sale. The unit must be checked out by trained service personnel, or by the manufacturer of the pump unit.

## 7. Maintenance / Care / Cleaning

The silicone pot integrated into the Pumpset ("Vacuum Seal" technology) ensures that no breast milk or air can get into the tube or Calypso unit. The closed system is so secure that it constitutes a barrier against cross contamination (infection).

The unit is maintenance-free; care of the unit and its accessories is limited to inspection and cleaning. There are no user-serviceable parts in the Calypso breastpump. With the exception of the power adapter (g), there are no spare parts.

### Visual inspection by the user before switch-on

- Is the power adapter (g) undamaged?
- Is the accessory clean, complete and intact?
- Is the accessory correctly attached?

Ignoring these checks, or the cleaning instructions, may lead to risk of infection, electric shock or damage to the unit.

### Cleaning / disinfection

- Calypso breastpump:  
Important: Unplug the unit from the power adapter before cleaning.  
As needed, wipe with a damp cloth, commercial detergent may be used. Do not put the unit into the dish-washer or into a washing-up bowl. It is not possible to clean the pump or the power unit. Attempting to do so may cause the unit to malfunction or to become contaminated – requiring suitable disposal (see section 10: Transport / Storage / Disposal). Calypso is not waterproof. It complies with Standard IP21.
- Pumpset: Please read the separate instructions for use, Section: Cleaning.

## 8. Warranty

For a period of one (1) year from the proven date of purchase or for a period of 400 hours of use (as long as the green LED display lights up), whichever is later, (the "Warranty Period"), Ardo warrants to the original purchaser of the unit against defects in material or workmanship. The purchase invoice serves as proof purchase and is the date on which the Warranty Period will begin. The warranty is limited to the original purchaser. Repair or replacement under this limited warranty is the sole and exclusive remedy of the original purchaser. Proof of purchase in the form of a received invoice or bill of sale evidencing that the Product is within the warranty period must be presented to obtain warranty service. If the unit is repaired or replaced under this Warranty, the Warranty Period will not be extended. The warranty will apply only if the instructions in the User Instructions are followed and only if Ardo medical AG accessories used. This warranty is provided free of charge. It does not affect any claims or legal or other rights of the customer under the applicable national laws governing the sale of consumer products. For warranty service contact 844-411-ARDO (2736).

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THE PRODUCT IS HEREBY DISCLAIMED. ARDO SHALL NOT BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL OR PUNITIVE DAMAGES OF ANY KIND FOR BREACH OF THE EXPRESS LIMITED WARRANTY ON THE PRODUCT OR ANY WARRANTY IMPLIED BY OPERATION OF LAW. OTHER THAN THE EXPRESS LIMITED WARRANTY SET FORTH ABOVE, THERE ARE NO OTHER WARRANTIES THAT ACCOMPANY THIS PRODUCT AND ANY ORAL, WRITTEN OR ANY OTHER REPRESENTATION OF ANY KIND IS HEREBY DISCLAIMED.

The warranty will be voided by: (1) unauthorized use or access to the unit; (2) if the unit has been opened; (3) improper handling contrary to the instructions for use; and (4) if the unit has been repaired, altered or modified by an entity other than Ardo. Nor does this warranty apply to unit defects or malfunctions caused by normal wear and tear, incorrect voltage, accident, fire or other causes not attributable to Ardo.

### NB

After 400 hours, the green LED display changes to orange. This means that the additional Guarantee has expired. The Calypso can however still be used without any reduction in performance.

## 9. Technical Information

9.1 Technical data			
Pump dimensions	190 x 130 x 76 mm	Classification in accordance with MDD93/42	Class 2a
Power adapter dimensions	75 x 46 x 31 mm	Power adapter input voltage	100 - 240 VAC ± 10 %
Pump weight	0.545 kg	Frequency	50 Hz / 60 Hz
Power adapter weight	0.150 kg	Regulated power adapter output	9 Vdc, 1.4 A
Double insulated protection class	Class II	Vacuum range	50 - 330 mbar
Protection level	BF	Permitted tolerance of the vacuum strength	± 5 %
Life time (excluding normal wear and tear)	800 hours	Operating temperature Relative humidity	41 °F to 104 °F (+5 °C to +40 °C) 20 - 95 %

9.2 Compliance	9.3 EMC Test records
In accordance with Medical Devices Directive MDD 93/42 EEC, EN 60601-1, EN60601-1-2, EN ISO 10079-1, EN 60601-1-6, EN60601-1-11, 2011/65/EU (RoHS), UL1431, ANSI/AAMI ES60601-1:2005, CSA C22.2 No. 68-09, CSA C22.2 No. 60601.1:2008	See Page 10  0123

## 9.4 Key to symbols

	Important information
	Warning: General risk points – check with the instructions for use
	Please note instructions for use
	Compliance with Directives MDD 93/42 EEC
	Manufacturer
	Unit type BF
	The unit is subject to the stipulations of WEEE Directive 2002/96/EC
	Protection class II
	IP21 drip-proof protection (Calypso)

	IP20 drip-proof protection (power adapter)
	Alternating current
	Direct current
	Temperature
	Air pressure
	Relative humidity
	Fragile, handle with care
	Keep packaging away from heat
	Keep packaging away from moisture

## 10. Transport / Storage / Disposal

For the transport and storage of your Calypso unit, we recommend using the original packaging. We further recommend separate storage protected from UV (e.g. sunlight/halogen lamps), heat and dust (Avoid storing or using the unit on a heater. Do not store or use the unit in a dusty environment. Failure to observe these instructions can lead to the malfunction of the unit).

Transport and storage conditions	
Temperature	41 °F to 104 °F (+5 °C to +40 °C)
Relative humidity	20 - 95 %
Air pressure	700 - 1060 hPa

## Disposal

Properly dispose of the device in accordance with local, state and federal regulations.

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## TEST REPORT

IEC / EN 60601-1-2

### Part 1-2: Electromagnetic Compatibility Requirements and Tests

Report reference No:	E1771-05-12			
Tested by:	B. Belegu			
Approved by management:	Dr. F. Stucki			
Date of issue:	13 September 2012			
Number of pages:	34 pages			
Testing laboratory:	QUINEL AG	 S Schweizerischer Prüfstellendienst Service Suisse d'essai Servizio di prova in Svizzera Swiss testing service		
Address:	Grundstr. 2	CH-6343 Rotkreuz		
Testing location:	Rotkreuz	Tel. 041-799 47 04		
Applicant's Name:	Ardo medical AG, Mr. Steiner Roland			
Address:	Gewerbestrasse 19,	CH-6314 Unterägeri		
Manufacturer:	Ardo medical AG			
Address:	Gewerbestrasse 19,	CH-6314 Unterägeri		
Test specification:	IEC 60601-1-2:2007, EN 60601-1-2:2007			
Standards:	IEC 60601-1-2:2007, EN 60601-1-2:2007			
Test procedure:	Type testing for Swiss and EU legal requirements			
Procedure deviation:	None			
Non-standard test method:	None			
Test-specification:	The used test setup fulfils the specification described in the relevant standards			
Test item description:	Breast pump			
Trademark:	Ardo medical AG			
Model and/or type reference:	Calypso			
Ratings:	9VDC, 1200mA			
Date of receipt of the test item(s):	12 September 2012			
Summary of testing:	Passed			
Applied standards:				
No	Title	Standard (up dated)	pages	Verdict
E1771-22-12	RF disturbances	CISPR 11:2009+A1:2010 EN 55011:2009+A1:2010	12	P
E1771-06-12	RF conducted disturbances	IEC 61000-4-6:2008 EN 61000-4-6:2009	2	P
E1771-03-12	Radiated electromagnetic field	IEC61000-4-3:2006+A1:2007+A2:2010 EN 61000-4-3:2006+A1:2008+A2:2010	4	P
E1771-08-12	Power frequency magnetic field	IEC 61000-4-8:2009 EN 61000-4-8:2010	2	P
E1771-04-12	Electrical fast transient burst	IEC 61000-4-4:2012 EN 61000-4-4:2004+A1:2010	2	P
E1771-02-12	Electrostatic discharge	IEC 61000-4-2:2008 EN 61000-4-2:2009	2	P
E1771-55-12	Surge	IEC 61000-4-5:2005 EN 61000-4-5:2006	2	P

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MEDICAL—GENERAL MEDICAL EQUIPMENT  
AS TO ELECTRICAL SHOCK, FIRE AND  
MECHANICAL HAZARDS ONLY  
IN ACCORDANCE WITH  
ANSI/AAMI ES60601-1 (2005, C1:09, A2:10),  
CAN/CSA C22.2 No.60601.1 (2008, R:2011)

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